


TIP UP LIFT CAR SEAT

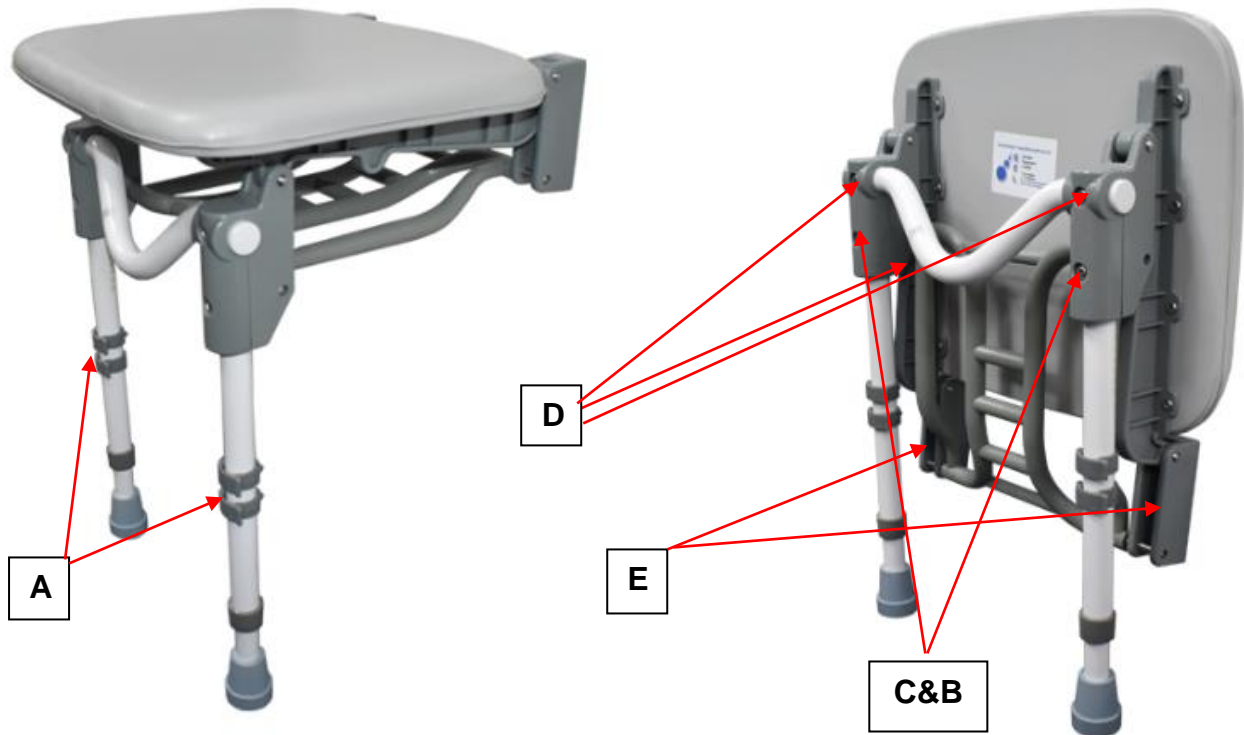

Intended uses

- To compensate for a disability by providing a seat for a person taking the lift.

Instructions for use

1. The seat can be folded upright out of the way when not in use. Be sure the latch holds the seat securely up.
2. To lower the seat, hold onto the cross bar between screws D, and pull out from the wall.

Cautions

- **DO NOT PULL ON THE LEGS TO LOWER OR RAISE THE SEAT.**
- The lift car tip up seat should be installed by a qualified professional.

The Lift Car Tip Up seats are not to be used to support more than 158kg / 348lb. / 24¾ st.

Be sure that the Lift Car Tip Up seat is securely in place before putting weight on it.

Instructions for installation

1.

These seats should be installed by a qualified professional, who will choose hardware appropriate for the specific installation. Note that each individual fixing should resist a shear force of 60 kg / 130 lb. and a pull-out force of 20 kg / 45 lb.

2.

Remove the screws that are labelled C in the drawing from each block (B) and loosen the screws that are labelled D.

3.

Slide the legs into the blocks and tighten screws D. Screws C can be left out at this stage.

4.

Establish the height of the seat, which can be adjusted from 460 to 590 mm / 18 to 23 inches as follows:

a.

With the seat in the horizontal position, position the rear brackets (E) against the lift car wall.

b.

To adjust leg length, remove the four pin clips (A) from the legs, determine the appropriate length, and replace the pin clips.

c.

Be sure that both legs are adjusted to the same height. A gap up to 5 mm / 3/16" under one leg is acceptable.

5.

Mark the lower securing holes through the rear brackets. All fixing holes in the rear bracket must be used. Remove the seat from the wall and drill and plug the wall. Installation fixing kit available, product code: DDA/ICS/KIT.

6.

Loosen the screws D and remove the legs from the seat.

7.

With the seat locked in the vertical position, position the rear brackets over the holes and fasten the seat to the lift car wall.

8.

Slide the legs back into the blocks and line up the leg securing holes. Replace and tighten screws C, and tighten screws D in the blocks.

Care and Cleaning

The seat can be cleaned with mild detergent and water.

